




Change of new FTP access data for Kulzer Milling Center in Dental Manager

1. Open the Control Panel in the Dental Manager SW

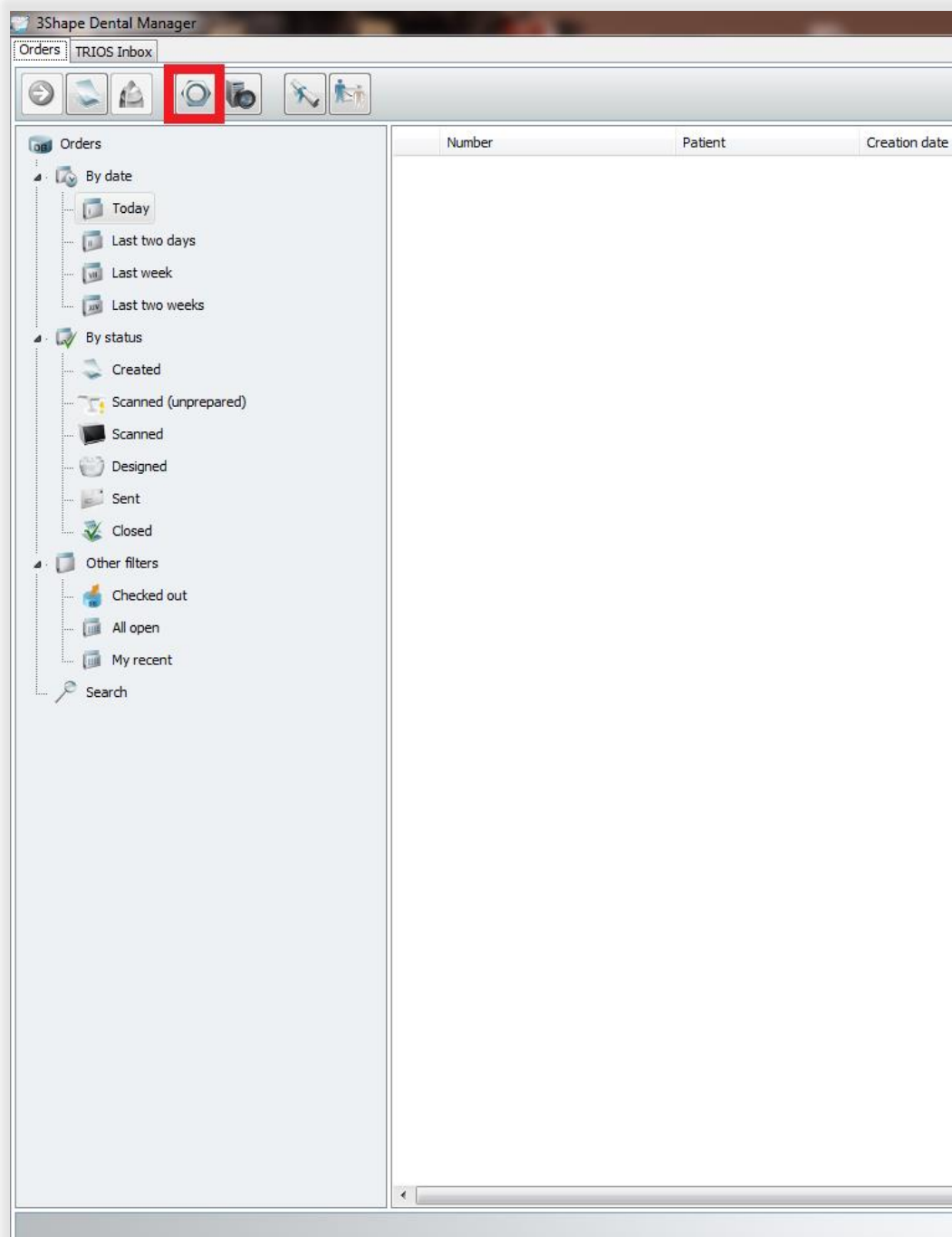



2. Open in the chapter „Tools“ the file subscription management





3. Click on Internet Update and execute the Update

Home Back Save Help

Tools > Subscription Management (Dongle)

3Shape Subscription Management

This page is used for updating the dongle connected to the PC with the 3Shape Subscription Management.

Applications

Dongle number: 2983591718

Applications enabled:

- Abutment Designer Module Save Limited DEMO (18 saves left)
- DentalDesigner Premium (54 saves left, 1 client points, expiry date 01.09.2017)
- DentalManager (1 client points, expiry date 01.09.2017)
- DentalManager TRIOS Inbox module (1 client points)
- Full Denture Design Module (10 saves left, 1 client points)
- Implant Bars/Bridge Design Module (9 saves left, 1 client points)
- Model Builder (3rd Party scans) (5 saves left, 1 client points)
- Model Builder (Impression scans) (19 saves left, 1 client points)
- Model Builder (TRIOS scans only) (43 saves left, 1 client points)
- Model Builder STL (expiry date 12.03.2016)
- Removable Design Module (24 saves left, 1 client points)
- ScanIt Impression (15 saves left, expiry date 01.09.2017)
- TRIOS Base (20 saves left, expiry date 01.09.2017)
- TRIOS Restoration (20 saves left, expiry date 01.09.2017)

Internet update Details

Modules and libraries

Dongle Service

Manual update



4. Open the „Site Settings-manufacturers”
 - (1. Heraeus Milling Center / old FTP Server setting) and
 - (2. click on “Import Sites from Server”.

Dental System Control Panel

Home Back Save Help

Site Settings > Manufacturers

Table

Heraeus Milling Center

1.

Alphabetically

Add
Copy
Delete
Move up
Move down

Recycle bin

No items
Restore all
Empty the Recycle Bin

Details

Total sites: 8 items

2.

Tasks

Import site(s) from server
Import sites from Excer document

Edit Site

Name: Heraeus Milling Center

Contacts settings

Address 1
Address 2
City
State
Zip code
Country: Not specified
Phone
Fax
Email
Web address
Contact person

3Shape Communicate Integration

TRIOS Direct Connection

ID settings

FTP Inbox settings

FTP IP address/URL name: 193.29.5.15
FTP port: 21
FTP user: HKGCADCAMP049
FTP password:
FTP directory: Germany

In order to use FTP with SSL support you need to install OpenSSL 0.9.8. or later

Validate material configuration ID when creating order
Enabled
Configuration ID: _____

Automatically send to manufacturer after scanning
Enabled

Automatically add Ship to different address
Enabled



5. After the Import has taken place you should find the new “Kulzer Milling Center” in your Site Settings. The new FTP Server is implemented!
Please save and close the Control Panel!

Dental System Control Panel

Home Back Save Help

Site Settings > Manufacturers

Table

Kulzer Milling Center

Alphabetically

- Add
- Copy
- Delete
- Move up
- Move down

Recycle bin

- No items
- Restore all
- Empty the Recycle Bin

Details

Total sites: 8 items

Tasks

- Import site(s) from server
- Import sites from Excel document

Edit Site

Name: Kulzer Milling Center

Contacts settings

Address 1
Address 2
City
State
Zip code
Country: Not specified
Phone
Fax
Email
Web address
Contact person

3Shape Communicate Integration

TRIOS Direct Connection

ID settings

FTP Inbox settings

FTP IP address/URL name: ftp.kulzer-dental.com

FTP port: 21

FTP user: hkgcadcamp049

FTP password: ●●●●●●

FTP directory: Germany

In order to use FTP with SSL support you need to install OpenSSL 0.9.8. or later

Validate material configuration ID when creating order

Enabled

Configuration ID

Automatically send to manufacturer after scanning

Enabled

Automatically add Ship to different address

Enabled

Contact in Germany

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau, Germany
cara-service@kulzer-dental.com
www.heraeus-kulzer.com/cara

New address as of 01 July 2017:

Kulzer GmbH
Leipziger Straße 2
63450 Hanau, Germany
cara-service@kulzer-dental.com
www.kulzer.com/cara