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GB Instructions for use

Photocuring Signum cre-active mass for individual characterisation of light-cured veneers and plastic teeth.

Composition of Signum cre-active colorfluids

Multifunctional methacrylic acid ester, highly dispersed silicon dioxide, cadmiumfree pigments

Composition of Signum cre-active transpaffluids

Dimethacrylate, silicon dioxide

Indications

Individual characterisations of light-cured veneers and plastic teeth, partial and full dentures and implant work – enamel cracks, calcification spots, imitation fillings, enamel wedges, coloured intermediate layer, abraded surfaces, fissures

Signum cre-active colorfluids
white, polar, umbra, corn, mango, caramel, maroon, black

Signum cre-active transpaffluids
TF1 – high viscosity, clear
TF2 – low viscosity, clear

Processing

The ready-to-use Signum cre-active mass can be applied directly onto the polymerised veneer material. Instead of using the margin component for thin veneer sections, Signum cre-active can be individually mixed and applied directly to the opaque layer. The Signum cre-active transpaffluids can be mixed in to reduce the colour intensity of Signum cre-active. Use pure Signum cre-active transpaffluid if highly transparent effects are desired. Always apply a layer of veneering material after application of Signum cre-active, T1 or T2. To repair a destroyed dispersion layer, roughen the surface with a diamond burr and brush on a thin but very moist layer of Signum liquid so that it extends beyond the site of repair. Allow 30 s for Signum liquid to react, do not polymerise separately.

<i>Device</i>	<i>Max. curable layer thickness</i>	<i>Polymeri-sation time</i>
<i>Heraflash®</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>UniXS</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>HILite power®</i>	<i>0,2 mm</i>	<i>90 s</i>

<i>Device</i>	<i>Max. curable layer thickness</i>	<i>Polymeri-sation time</i>
<i>Heraflash®</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>UniXS</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>HILite power®</i>	<i>0,2 mm</i>	<i>90 s</i>

<i>Device</i>	<i>Max. curable layer thickness</i>	<i>Polymeri-sation time</i>
<i>Heraflash®</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>UniXS</i>	<i>0,2 mm</i>	<i>90 s</i>
<i>HILite power®</i>	<i>0,2 mm</i>	<i>90 s</i>

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